# OPERATION GUIDE QUALITY

Attention: Carefully read this manual  
before operating the Olsen's equipment!

**Olsen**  
Equipment made to last

# Index

1 - Introduction .....	5
2 - Package Contents .....	5
3 - Equipment Presentation .....	5
3.1 - Standard Configuration .....	5
3.1.1 - Dental Chair .....	5
3.1.2 - Operating Light .....	5
3.1.3 - Delivery System .....	5
3.1.4 - Water Unit .....	5
3.2 - Optional Items .....	6
3.3 - Quality Models .....	6
3.3.1 - Quality Flex .....	6
3.3.2 - Quality EX .....	6
3.3.3 - Quality Cross Flex .....	6
4 - Parts Identification .....	7
5 - Equipment Operating and Description .....	7
5.1 - Before Turning On the Equipment .....	7
5.2 - Turning On the Equipment .....	7
5.2.1 - Delivery System .....	7
5.2.2 - Electric Panel .....	7
5.3 - Base and Progressive Pedal Controls (Operation) .....	8
5.3.1 - Remote Foot Pedal Commands (Opcional) .....	8
5.4 - Working Table .....	8
5.4.1 - Quality Working Table .....	8
5.4.2 - Working Table Quality Cross Flex .....	10
5.4.3 - Flex Arm .....	11
5.5 - Operating Lights .....	11
5.5.1 - Concept LED .....	11
5.5.2 - Concept Plus (Optional) .....	11
5.5.3 - Premium LED (Optional) .....	11
5.6 - Water Unit .....	12
5.7 - Assistant Module (Optional) .....	12
5.8 - Dental Chair .....	12
5.8.1 - Work Position .....	13
5.8.2 - Zero Position .....	13
5.8.3 - Headrest .....	13

5.8.4 - Swivel Right Armrest (Optional) .....	13
6 - Instruments Operation.....	14
6.1 - Pneumatic Handpiece Couplings .....	14
6.2 - Optical Fiber Handpiece (Optional) .....	14
6.3 - Electrical Micromotor (Optional).....	14
6.3.1 - Electrical Micromotor Basic Adjustments.....	14
6.3.2 - Brushless Electrical Micromotor - Built in (Optional) .....	15
6.3.3 - Brushless Electrical Micromotor - Side Panel (Optional) .....	15
6.4 - Scaler.....	16
6.4.1 - Standard Scaler.....	16
6.4.2 - Scaler with LED Light (Optional) .....	16
6.4.3 - Scaler Adjustments .....	17
6.4.4 - Scaler Tips .....	17
6.5 - Curing Light (Optional) .....	17
6.6 - 3-Way Syringe.....	18
6.7 - Venturi Saliva Ejector.....	18
6.8 - High Power Saliva Ejector Vórtice (Optional).....	18
6.9 - Vacuum Pump Adaptor (Optional) .....	18
6.10 Prophylaxis System.....	19
6.10.1 - Bicarbonate Jet (Optional) .....	19
6.10.2 - Prophy-Jet System (Optional).....	19
6.11 - Air Jet System (Optional).....	19
6.12 - X-Ray Viewer.....	19
6.13 - Thermo Comfort System (Optional).....	19
6.14 - Anti-Stress System (Optional).....	20
6.15 - Monitor Holder (Optional).....	20
6.16 - Cart System (Optional).....	20
7 - Symbology .....	21
8 - Olsen Accredited Technical Assistance Network.....	22
9 - Warranty Terms.....	23
10 - Message from the President.....	24

# 1 - Introduction

Congratulations on choosing Olsen equipment!

You have acquired high-tech equipment developed to offer the best performance in dental procedures ensuring ergonomics and comfort, for both professional and the patient.

The purpose of this guide is to present instructions for operating the dental equipment of the Quality line. The images are merely illustrative and do not represent the configuration of models or the availability of items and accessories.

The *User Manual Dental Line* is supplementary to this guide and should be read before operating the equipment. It is provided in digital format (PDF) along with the order confirmation for your equipment. The user manual presented the descriptive and technical information, installation instructions, maintenance, cleaning, precautions of use and regulatory guidelines.

# 2 - Package Contents

## Standard Items:

- 1 Dental Chair Quality
- 1 Delivery Unit Quality
- 1 Operating Light
- 1 Packing Support
- 1 Dental Stool
- 1 Accessories Box
- 1 Quality Quick Operation Guide

## Optional Items:

- Handpieces
- Separated Junction Box
- Cart System
- Additional Stool
- Monitor Holder

**Note:** items availability in the packaging may change in the case of the customer order.

# 3 - Equipment Presentation

## 3.1 - Standard Configuration

### 3.1.1 - Dental Chair

- 3 programmable Work Positions
- Automatic Zero Position
- Joystick and Commands on the Chair Base
- Integrated Junction Box
- Multi-Articulating Headrest
- Swivel Right Armrest

### 3.1.2 - Operating Light

- Evolution Plus LED 30.000 Lux
- On/Off on the Base, Pedal or Control Panels
- Side Handles
- Multifaceted Mirror

### 3.1.3 - Delivery System

- Coupling for Low and High-speed Handpiece
- 3-Way Syringe
- Stainless Steel Tray
- Flex Arm with Pneumatic Lock and Capacitive Sensor
- X-Ray Viewer

### 3.1.4 - Water Unit

- Swivel at 90°
- Translucent Spittoon Bowl
- Cup Filler
- Venturi Saliva Ejector

## 3.2 - Optional Items

Commands on the Remote Foot Pedal  
Emergency Stop Button  
Leather Upholstery  
Anti-Stress System  
Automatic Vapor Drain  
Cart System  
Additional Handpiece Position  
Air Jet System  
Optical Fiber Handpiece Coupling  
LED Curing Light on the Delivery System  
Electrical Micromotor Brushless  
Electrical Micromotor  
Scaler with LED Light  
Standard Scaler  
Prophylaxis System with Bicarbonate Jet  
Midwest Couplings  
Blue Touch Panel  
Prophy-Jet System

Thermo Comfort System  
Monitor Holder  
Premium LED Operating Light 8.000 to 30.000 Lux  
Concept Plus Operating Light 8.000 to 30.000 Lux  
Bowl Flush Water Control and Cup Filler with Timer  
Assistant Module  
Additional 3-Way Syringe on the Water Unit or  
Assistant Module  
Vacuum Pump Adaptor Set  
High Power Vórtice Saliva Ejector  
Quick Coupler for Saliva Ejectors and/or Vacuum  
Pump  
Additional Prophy-Jet System on the Assistant  
Module  
Additional LED Curing Light on the Assistant  
Module  
Porcelain Spittoon Bowl  
Junction Box

**Note:** optional items are defined on the customer's order.

## 3.3 - Quality Models

**Quality Flex**

**Quality EX**

### 3.3.1 - Quality Flex

It presents the standard items of the Quality line (section 3.1).

### 3.3.2 - Quality EX

It presents, in addition to the standard items of the Quality line (section 3.1):

Additional Handpiece Position  
Standard Scaler  
Prophylaxis System with Bicarbonate Jet  
Blue Touch Panel  
Bowl Flush Water Control and Cup Filler with Timer

**Quality Cross Flex**

### 3.3.3 - Quality Cross Flex

It presents, in addition to the standard items of the Quality line (section 3.1):

Cross Flex Working Table with Jointed Stainless-Steel Tray  
Additional Handpiece Position  
Bowl Flush Water Control and Cup Filler with Timer

## 4 - Parts Identification

- A - Delivery System
- B - Electrical Panel
- C - Progressive Pedal
- D - Working Table
- E - Assistant Module
- F - Operating Light
- G - Dental Chair
- H - Water Unit

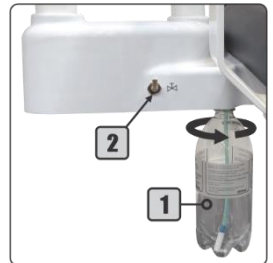
## 5 - Equipment Operating and Description

### 5.1 - Before Turning On the Equipment

- 1° - Check if the equipment is properly installed in accordance to the instructions in the *chapter 6 (Installation)* of the *User Manual Dental Line*;
- 2° - Turn On the air compressor that supplies the equipment;
- 3° - Open the main water supply valve of the equipment;
- 4° - Turn On the equipment's power circuit breaker.

### 5.2 - Turning On the Equipment

- 1° - Turn the On/Off switch (3), its LED indicator will light, indicating that the equipment is on;
- 2° - Activate the "Seat Up" command (*section 5.3*) on the chair base until the seat reaches its maximum height limit;
- 3° - Close the pressure valve (2);
- 4° - Make sure the water tank (1) is full. If necessary, fill it with drinkable water up to the limit indicated in tank and reattach it again. If you wish to place prophylactic products (low concentration);
- 5° - Open the pressure valve (2) and check if the water tank (1) is well attached and free of leaks;
- 6° - Before starting the use equipment, verify the operation of all instruments and commands available and perform the cleaning and sterilization of the instruments (*chapter 7 of the User Manual Dental Line - Cleaning and Disinfection*).

#### 5.2.1 - Delivery System

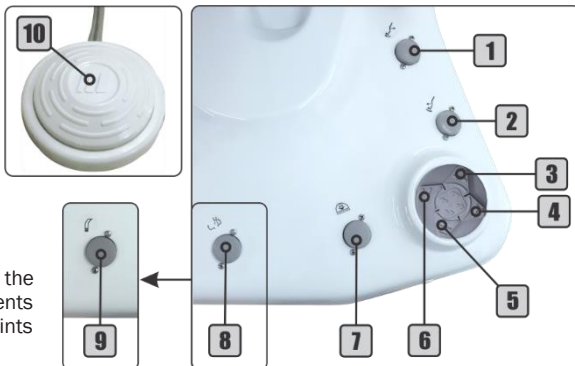
- 1 - Water Tank
- 2 - Pressure Valve

#### 5.2.2 - Electric Panel

- 3 - On/Off Switch with LED Indicator
- 4 - Fuse Holder

## 5.3 - Base and Progressive Pedal Controls (Operation)

- 1 - Zero Position
- 2 - Work Position
- 3 - Backrest Up (↗)
- 4 - Seat Down (↓)
- 5 - Backrest Down (↙)
- 6 - Seat Up (↑)
- 7 - On/Off Operating Light
- 8 - Spittoon Bowl Flush
- 9 - On/Off Vórtice Saliva Ejector
- 10 - Progressive/Activation Pedal


To activate each command simply press the respective button or in the case of the movements of the chair, press the joystick at the points indicated in the side image (3 to 6).

In case of the equipment has the high-power Vórtice saliva ejector, the spittoon bowl flush command (8) will be placed on the water unit, and the command for Vórtice ejector (9) will be available on the chair base.

**To activate the handpieces:** press the progressive/activation pedal (10).

### 5.3.1 - Remote Foot Pedal Commands (Opcional)

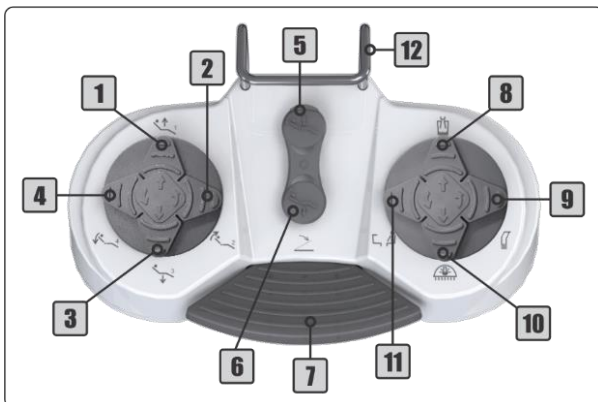
- 1 - Seat Up (↑)/Position 1
- 2 - Backrest Up (↗)/Position 2
- 3 - Seat Down (↓)/Position 3
- 4 - Backrest Down (↙)/Position 4
- 5 - Work Position
- 6 - Zero Position
- 7 - Propulsion/Activation Pedal
- 8 - Cup Filler
- 9 - On/Off Vórtice Saliva Ejector
- 10 - On/Off Operating Light
- 11 - Spittoon Bowl Flush
- 12 - Metal Handle

## 5.4 - Working Table

### 5.4.1 - Quality Working Table

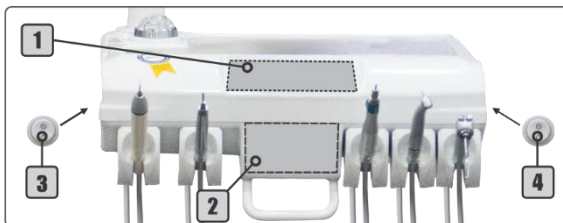- 1 - Command Panel
- 2 - Instruments Control Panel
- 3 - On/Off Optical Fiber/Air Jet System
- 4 - On/Off Thermo Comfort System

The layout of the controls and panels may vary according customer order and available handpieces.


Some functions indicated on control panel may not be available according to the equipment configuration.


**Command Panel Models (1):**

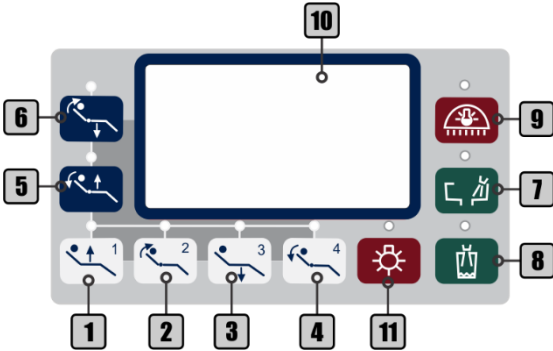**Blue Touch Panel with X-Ray Viewer**


**X-Ray Viewer**

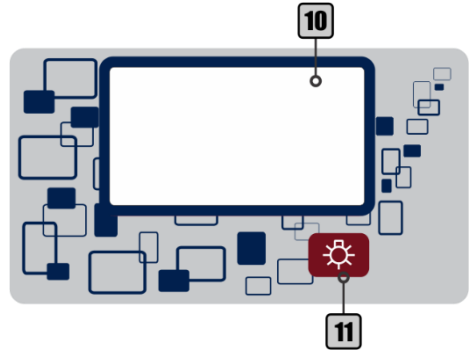- 1 - Seat Up/Position 1
- 2 - Backrest Up/Position 2
- 3 - Seat Down/Position 3
- 4 - Backrest Down/Position 4
- 5 - Work Position

- 6 - Zero Position
- 7 - Spittoon Bowl Flush
- 8 - Cup Filler
- 9 - On/Off Operating Light
- 10 - X-Ray Viewer
- 11 - On/Off X-Ray Viewer

**Instruments Control Panel (2):**

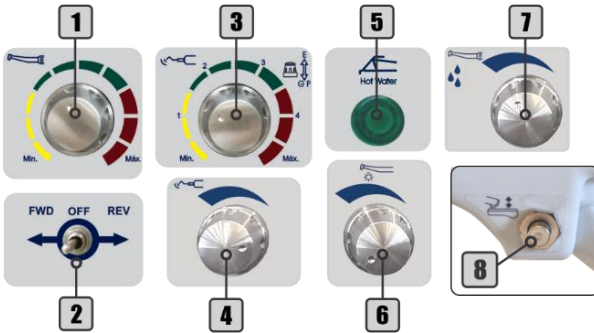- 1 - Electrical Micromotor Speed Control
- 2 - Electrical Micromotor Direction Control
- 3 - Scaler Power Control and Function Selector - General-Perio/Endo
- 4 - Scaler Water Control
- 5 - On/Off Thermo Comfort LED
- 6 - Optical Fiber Water Control
- 7 - Brushless Micromotor Water Control
- 8 - Pneumatic Brake Valve\*

\* The pneumatic brake valve of the flex arm is an optional item that is available when the system does not have the capacitive system.

## 5.4.2 - Working Table Quality Cross Flex

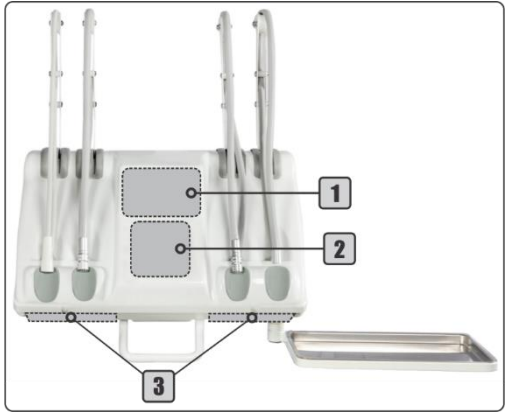
- 1 - Top Panel Cross
- 2 - Command Panel
- 3 - Instruments Control Panel


The layout of the controls and panels may vary according customer order and available handpieces.

Some functions indicated on control panel may not be available according to the equipment configuration.


### Top Panel Cross (1):

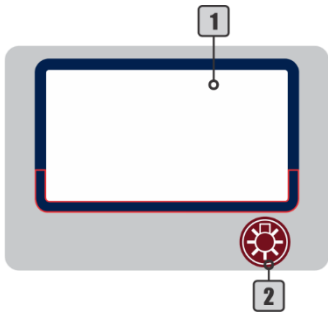- 1 - X-Ray Viewer
- 2 - On/Off X-Ray Viewer

### Command Panel (2):

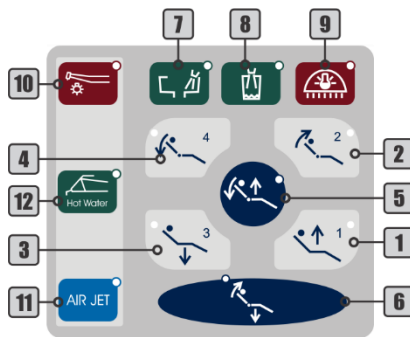- 1 - Seat Up/Position
- 2 - Backrest Up/Position
- 3 - Seat Down/Position
- 4 - Backrest Down/Position 4
- 5 - Work Position
- 6 - Zero Position
- 7 - Spittoon Bowl Flush
- 8 - Cup Filler
- 9 - On/Off Operating Light
- 10 - On/Off Optical Fiber Light
- 11 - On/Off Air Jet System
- 12 - On/Off Thermo Comfort System

### Instruments Control Panel (3):

- 1 - Electrical Micromotor Speed Control
- 2 - Electrical Micromotor Direction Control
- 3 - Scaler Power Control and Function Selector - General-Pério/Endo
- 4 - Scaler Water Control
- 5 - Optical Fiber Water Control

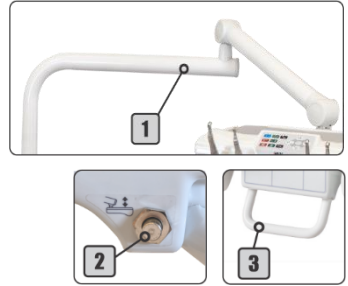
These controls are provided below the instrument to which they refer.

### 5.4.3 – Flex Arm

Features 2 articulations and pneumatic lock that are disabled by touching the capacitive handle (3) or by activating the pneumatic valve (2), allowing the working table movement.

**Do not move the table upright without disabling the pneumatic brake. This may damage the flex arm locking system.**

## 5.5 - Operating Lights

### 5.5.1 - Concept LED

**To turn On/Off the operating light:** press the operating light control (1) on the chair base or with pedal or command panel.

### 5.5.2 - Concept Plus (Optional)

**To activate the operating light:** press the operating light command (1) on the chair base (section 5.3) or on the command panel (section 5.4) to enable sensor (3) and control lever (2).

**To turn On/Off the operating light:** pass the hand in front of the sensor (3) approximately 5 cm away or turn to left or right the control lever (2).

**To change the light intensity:** hold your hand in front of the sensor (3) or hold the control lever (2) to the left or to the right until the operating light shows the desired intensity.

### 5.5.3 - Premium LED (Optional)

**To activate the operating light:** press the operating light command (1) on the chair base (section 5.3) or on the command panel (section 5.4) to enable sensor (5) and control knob (4).

**To turn On/Off the operating light:** pass the hand in front of the sensor (5) approximately 5 cm away or press the control knob (4).

**To change the light intensity:** hold your hand in front of the sensor (5) or turn the control knob (4) until the operating light shows the desired intensity.

## 5.6 - Water Unit

The water unit may display water controls for the bowl flush (1) and cup filler (2) in Water Unit side, in commands base or in remote foot pedal, according to the equipment configuration. If it has the cup filler, the control may be manual or temporized and the waterspout is removable.

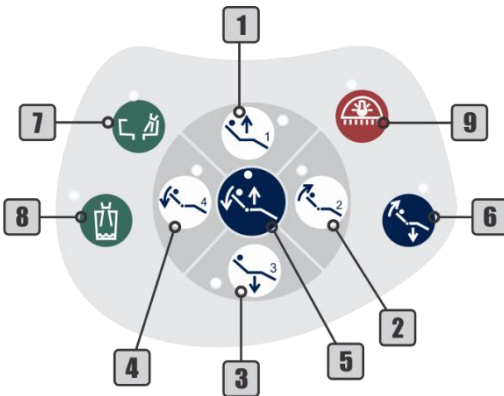
To control the flow of water for cup filler (2) and spittoon bowl (1), the water unit provides a needle valve (3).

**!** The availability of commands and instruments on water unit or assistant module may vary according customer order and available handpieces.

## 5.7 - Assistant Module (Optional)

When the equipment has assistant module, the instruments of the water unit are transferred to the module, facilitating access for the operator, through the extension arm.

- 1 - Seat Up/Position 1
- 2 - Backrest Up/Position 2
- 3 - Seat Down/Position 3
- 4 - Backrest Down/Position 4
- 5 - Work Position
- 6 - Zero Position
- 7 - Spittoon Bowl Flush
- 8 - Cup Filler
- 9 - On/Off Operating Light

## 5.8 - Dental Chair

- 1 - Seat Up (↑)
- 2 - Seat Down (↓)
- 3 - Backrest Down (↙)
- 4 - Backrest Up (↘)

To move backrest and seat: use the joystick buttons 1 to 4, located on the chair base (section 5.3) or panel Blue Touch (optional).

### 5.8.1 - Work Position

To save a work position:

- 1° - Press the Zero Position key;
- 2° - Adjust the backrest to the desired position;
- 3° - Adjust the seat to the desired position;
- 4° - Press the Work Position key for 5 seconds. The equipment will emit 2 long beeps;
- 5° - Press command 1 (*Seat Up*) while the equipment beeps; The equipment will emit 2 short beeps to confirm the procedure;

**Note:** the time to record the working position is only 2 seconds between steps 4 and 5, then when you hear the first long beep (step 4), quickly press one of the available commands (1 to 3) to record the working position. If none of the available commands are pressed during this time, the equipment will emit 3 short beeps, canceling the operation.

- 6° - Press the Zero Position key.

Repeat the previous steps by replacing the command 1 by the command 2 (*Backrest Up*) to record the working position 2 or by the command 3 (*Seat Down*) to save the working position 3.

To perform a working position:

- 1° - Press the Work Position key; The equipment will emit a short beep;
- 2° - Press the command for the desired saved working position (1 to 3); The equipment beeps twice confirming the operation.

The dental chair moves the backrest and seat simultaneously to the previously saved position and lights the operating light at the last intensity used.

**Note:** after pressing the Work Position key, the time for position selection is 4 seconds. If no key is pressed within this time, the equipment cancels the operation.

### 5.8.2 - Zero Position

Press the Zero Position key to automatically adjust the seat to the lowest position and backrest in the highest position simultaneously, to facilitate patient entry and exit.

### 5.8.3 - Headrest

To adjust the headrest (2) position: turn the knob (1) counterclockwise, adjust the headrest (2) in the desired position. Turn the knob (1) clockwise to lock the headrest in the set position.

To adjust the height of the headrest (3): move the bar (3) until you reach the position. The headrest bar allows adjustment with up to 17 cm from the backrest.

### 5.8.4 - Swivel Right Armrest (Optional)

To move the swivel armrest: press the trigger (1) and push the armrest down.

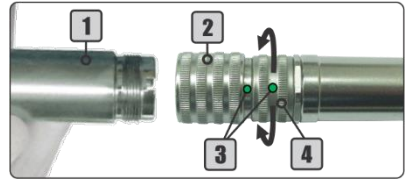
## 6 - Instruments Operation

### 6.1 - Pneumatic Handpiece Couplings

Before starting the operation connect the handpiece (1) to the coupling by attaching it to the coupling ring (2), ensuring that it is properly connected and that there is no leakage of air or water.

**To activate the rotation:** press the propulsion/activation pedal (5). The higher the pressure applied to the pedal, the more speed in the handpiece, until reaching the pressure limits of the equipment.

**To adjust the spray:** turn the adjusting ring (4) until the desired volume of water. To adjust the maximum flow of water, bring the spray adjustment indicators (3) closer to each other. To decrease the flow of water, rotate the ring so it distances one indicator from the other. The completely opposite points indicate the minimum water regulation for the spray.




### 6.2 - Optical Fiber Handpiece (Optional)

Before starting the operation connect the optical fiber handpiece to the coupling by attaching it to the coupling ring (1), ensuring that it is properly connected and there is no leakage of air or water.

**To activate the rotation:** press the propulsion/activation pedal (2). The higher the pressure applied to the pedal, the more speed in the handpiece, until reaching the pressure limits of the equipment.

**To turn the coupling's light On/Off:** use the On/Off optical fiber light (3) on the command panel (section 5.4) or the On/Off key (4) on the left side of the working table (section 5.4).

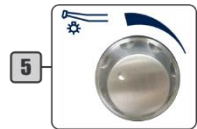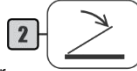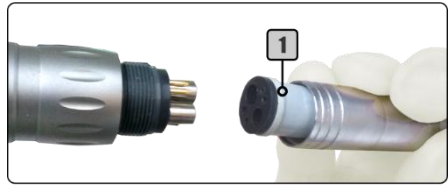**To adjust the spray:** use the optical fiber water control knob (5) on the instruments control panel.

### 6.3 - Electrical Micromotor (Optional)

Before starting the operation, connect the handpiece to the electrical micromotor (2).

**To activate the rotation:** press the progressive/activation pedal (1).

#### 6.3.1 - Electrical Micromotor Basic Adjustments

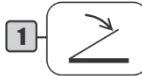
**To adjust the speed:** turn the control knob (3) until the desired speed is reached.

**To change the rotation direction:** use the rotation direction control (4).


**FWD:** clockwise;

**OFF:** Off;

**REV:** counterclockwise.

### 6.3.2 - Brushless Electrical Micromotor - Built in (Optional)

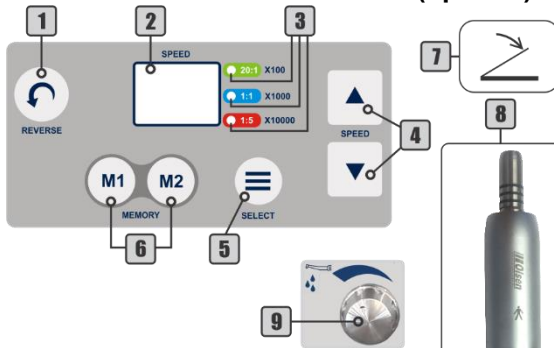Valor no Display	Velocidade (RPM)		
	20:1 (x100)	1:1 (x1.000)	1:5 (x10.000)
01	100	2.000	10.000
02	200	3.000	20.000
03	300	4.000	30.000
04	400	5.000	40.000
05	500	5.000	50.000
06	600	6.000	60.000
07	700	7.000	70.000
08	800	8.000	80.000
09	900	9.000	90.000
10	1.000	10.000	100.000
11	1100	11.000	110.000
12	1200	12.000	120.000
13	1300	13.000	130.000
14	1400	14.000	140.000
15	1.500	15.000	150.000
16	1.600	16.000	160.000
17	1.700	17.000	170.000
18	1.800	18.000	180.000
19	1.900	19.000	190.000
20	2.000	20.000	200.000
21	2.100	21.000	
22	2.200	22.000	
23	2.300	23.000	
24	2.400	24.000	
25	2.500	25.000	
26		26.000	
27		27.000	
28		28.000	
29		29.000	
30		30.000	
31		31.000	
32		32.000	
33		33.000	
34		34.000	
35		35.000	
36		36.000	
37		37.000	
38		38.000	
39		39.000	
40		40.000	

Before starting the operation, connect the handpiece to the electrical micromotor (8).

**To activate the rotation:** press the propulsion/activation pedal (7).

**To select the transmission:** use the Select key (5). The selection is indicated by LED's (3): 20: 1; 1: 1 and 1: 5.

To reverse the rotation: use the Reverse key (1).

**To adjust speed:** use the adjustment key (4) The speed relation of each transmission with the value of the display (2) is indicated in the side table.

Example:

Display value: 15

Selected gear ratio: 1:1 Factor: x1.000

Rotation speed: 15 x 1000 = 15.000 rpm

**To activate memory:** press the desired Memory key (6). The display will show the speed and the LED will indicate the present transmission.

**To record memory:** select the transmission and adjust the speed of rotation to store. Press and hold one of the Memory key (6) until the system emits 2 beeps, indicating that the memory has been recorded.

To save the second available memory repeat the same procedure.

**To adjust the electrical micromotor water:** use the water control knob (9) on the instruments control panel.


### 6.3.3 - Brushless Electrical Micromotor - Side Panel (Optional)

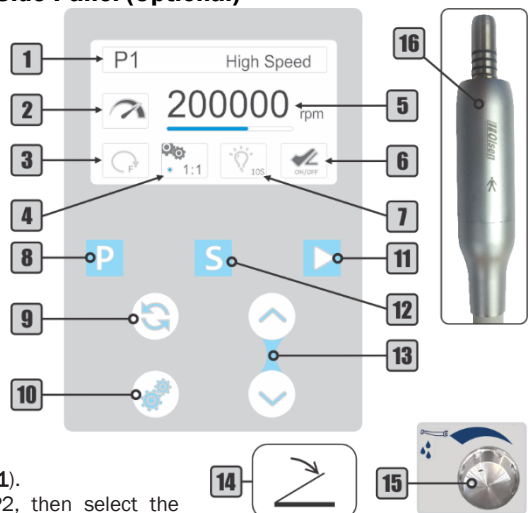Before starting the operation, connect the handpiece to the electrical micromotor (16).

**To activate the rotation:** press the propulsion/activation pedal (14) or the Manual Start key (11).

**To select a preset program:** use the Program key (8). There is 7 programs available, according the table below:

Program	Default settings
P1 My favorite1	
P2 My favorite2	
P3 Tooth prep (H)	1:5 200.000 rpm
P4 Tooth prep (L)	1:5 20.000 rpm
P5 Contra angle	1:1 40.000 rpm
P6 Straight	1:1 40.000 rpm
P7 Polishing	1:1 5.000 rpm

The selected program is shown on the display (1).

**To record program:** select program P1 or P2, then select the

desired gear ration and adjust the speed. Press and hold the Program key (8) until the confirmation of the system.

**To adjust speed:** use the adjustment keys (13) to increase or decrease speed. Please see the table below for speed range for each gear ration value.

Gear ratio	Speed range (rpm)
1:5	10.000 - 200.000
1:1	2.000 - 40.000
4:1	500 - 10.000
10:1	200 - 4.000
16:1	100 - 2500
20:1	100 - 2000

The selected program is shown on the display (5).  
**To select the gear ratio:** use Gear Ratio key (10). There are the following values available: 1:5, 1:1, 4:1, 10:1, 16:1 and 20:1.

The selected gear ratio is shown on the display (4).

**To change the rotation direction:** use the Direction key (9).

Reverse

Forward

The selected direction is shown on the display (3).


**To adjust the micromotor water:** use the water control knob (15) on the instruments control panel.

**To access setup menu:** press and hold the Setting key (12).

Use the Setting key (12) to select the desired option.

**To adjust LED options:** use the adjustment keys (13) to set the parameter.

LED Off	LED On without delay	LED On with 10s delay

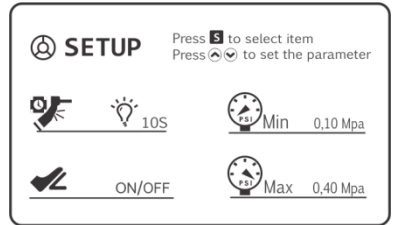**To adjust pneumatic activation method:** use the adjustment keys (13) to set the parameter.

	ADJUST
Micromotor performs speed according to the pressure on the pedal (14)	
	ON/OFF
When pedal (14) is pressed micromotor performs the maximum speed adjusted	

**To adjust input pneumatic pressure:** use the adjustment keys (13) to set the parameter.

	0,10 Mpa
Set minimum activation pressure from 0.03 to 0.10 MPa	
	0,40 Mpa
Set maximum input pressure from 0.20 to 0.40 MPa	

**Note:** the micromotor system preset by the factory according to the internal equipment pressure. It is not recommended to change the pressure parameters.

## 6.4 - Scaler

### 6.4.1 - Standard Scaler

To start the use of scaler it is necessary to first install the tip that will be used, according to the instructions in section 6.4.2.

**To activate the scaler:** it is necessary to remove the scaler from the support and press the propulsion/activation pedal (6).

Observe the settings for scaler in section 6.4.3.

The standard scaler is not removable.

### 6.4.2 - Scaler with LED Light (Optional)

For the use of scaler, it is necessary to connect the transducer (1) to the coupling (3) and install the tip on the transducer (1).

**To attach the transducer (1) to the coupling:** align the adjustment indicators (2) of the two parts and carefully insert them.

**To install the scaler tips:** insert the tip into the transducer (1) by threading it carefully, then insert the Torque wrench (4) into the tip and then rotate it clockwise until it is tight. To remove the tip, turn it counterclockwise with the Torque wrench (4).

**To install a file adapter (not supplied with scaler):** attach the adapter to the transducer (1) and screw it

gently. Insert the file into the tip of the adapter, thread the tip and then use the Endo wrench (5) to tighten. To remove the adapter, use the Endo wrench (5) to loosen it and then carefully unscrew.

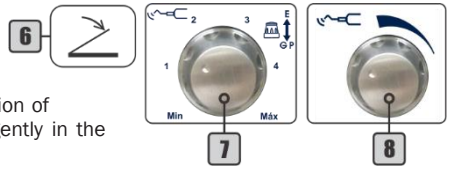
### 6.4.3 - Scaler Adjustments

**To activate the scaler:** push the propulsion/activation pedal (6).

**To adjust the vibration power:** turn the control knob (7) until the desired power is reached.

**To toggle functions:** push the control knob (7) in direction of the table for "GP" function (General - Perio). Pull the knob gently in the opposite direction of the table for "E" (Endo) function.

**To adjust the water:** use the control button (8) on the instrument control panel.

### 6.4.4 - Scaler Tips

The tips GD1, PD1 and PD3 accompany the scaler. The power and application information for each tip is available in the table below.

Tip	Code	Function	Power	Indication of use
	GD1	G	1-10	Removal of calculus and plates in the supragingival, interdental and tooth surfaces.
	PD1	P	1-10	Removal of calculus subgingival
	PD3	P	1-6	Removal of deep calculus subgingival

**Note:**

- 1 - Do not use the tip on features other than those specified by the manufacturer.
- 2 - Do not exceed the power limit for each tip. That could damage the tip and the scaler.

### 6.5 - Curing Light (Optional)

Before starting the curing light use, install the tip (1) and the light hood (2).

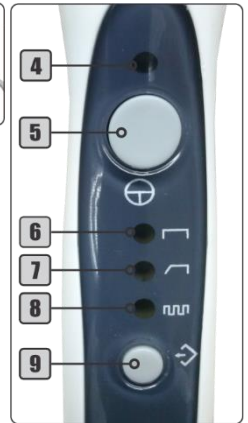
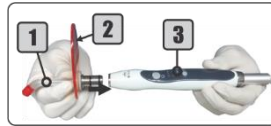
**To install the tip:** first, attach the light hood (2) to the tip and then attach it to the curing light unit (3). The tip must be pushed all the way to the fitting end.

**To activate/deactivate the curing light:** press the On/Off button (5) twice. The green LED (4) indicates that the curing light is operating. The curing light unit will run for 20 seconds and will turn Off automatically.

**To toggle the operating mode:** each tap on the selection button (9) switches the operation mode. Press the selection button (9) until the orange LED indicates the desired mode.

The curing light operates in the following modes:

- Continuous (6): turns on the maximum power for 20 seconds;
- Ramp (7): turns on the minimum power, gradually increasing. In 5 seconds reaches its maximum power until the cycle of 20 seconds is finished;
- Pulse (8): the device blinking for 20 seconds.


## 6.6 - 3-Way Syringe

Before beginning the use of the 3-way syringe (5), connect the syringe tip (3) by pressing the locking ring (4) into the correct fit.

**To emit air jet:** press the air button (1).

**To emit water jet:** press the water button (2).

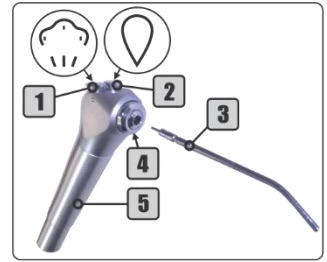**To emit spray jet:** press simultaneously the air and water buttons (1 and 2).


## 6.7 - Venturi Saliva Ejector

Before starting the operation with the Venturi saliva ejector (2), insert the cannula into the  $\varnothing$ 6,3 or 9,5 mm adapter (3).

**To activate the suction:** remove the ejector (2) from the holder (1).

**To deactivate the suction:** place the ejector in its holder (1).

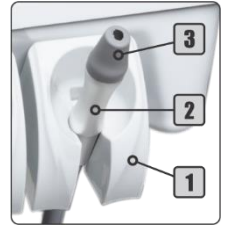The cannula adapter (3) is removable for cleaning.


## 6.8 - High Power Saliva Ejector Vórtice (Optional)

Before starting the operation with the Vórtice saliva ejector (2), insert the cannula into the  $\varnothing$  6,3, 9,5 or 11 mm adapter (3).

**To enable/disable suction:** use the On/Off Vórtice saliva ejector button (1) on the chair base.

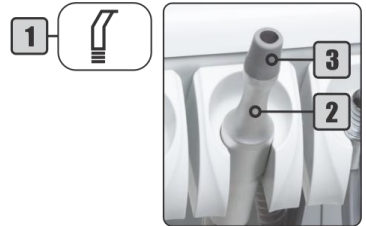The cannula adapter (3) is removable for cleaning.

## 6.9 - Vacuum Pump Adaptor (Optional)

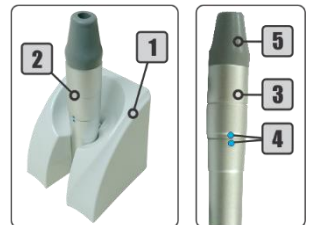
Before starting the operation with the Vacuum Pump saliva ejector (2), insert the cannula into the  $\varnothing$  6,3, 9,5 or 11 mm adapter (5).

**To activate suction:** remove the vacuum pump suction (2) from its holder (1).

**To adjust suction flow:** to decrease the suction flow, turn the adaptor's body (3) away from the indicators (4). The more distant, the smaller the suction. For the maximum suction flow, align the indicators (4).

**To deactivate the suction:** place the vacuum pump in its holder (1).


The cannula adapter (5) is removable for cleaning.

## 6.10 Prophylaxis System

### 6.10.1 - Bicarbonate Jet (Optional)

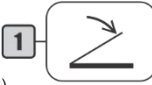Before starting use of the bicarbonate jet, its necessary deposit bicarbonate sodium in the reservoir (4), check for moisture in the filter (8) and connect the bicarbonate jet handpiece (3) to its coupling (2).

**To fill the bicarbonate reservoir:** open the reservoir (4) by turning the cover (6) anticlockwise and deposit the bicarbonate sodium until to the red line (5). Close the reservoir.


**To remove condensed water in the filter:** press the valve (7) on the base of the filter (8) to drain the water.

**To activate/deactivate the bicarbonate jet:** press the progressive/activation pedal (1). The more pressure on the pedal, the greater the volume of the jet emitted by the handpiece. until reaching the pressure limits of the equipment.

**To adjust the spray:** see section 6.1.

**Clean the sodium bicarbonate tank (4) at the end of each procedure.**
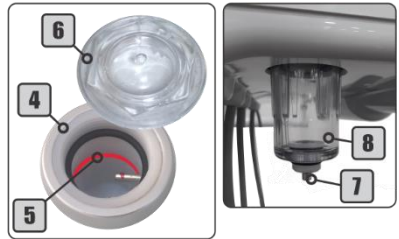
### 6.10.2 - Prophy-Jet System (Optional)

Before starting the Prophy-Jet system (3) use, deposit up to 15g of sodium bicarbonate for dental use sodium bicarbonate reservoir (4) and attach the Prophy-Jet to the pneumatic coupling available on the assistant module or pneumatic coupling (2) for low-speed handpiece in the working table.

**To activate/deactivate:** press the progressive/activation pedal (1). The higher the pressure applied to the pedal, the higher the air volume of the jet emitted by the Prophy-Jet system, until reaching the pressure limits of the equipment.

**Clean the sodium bicarbonate tank (4) at the end of each procedure.**

in the

## 6.11 - Air Jet System (Optional)

**To activate/deactivate the Air Jet system:** press the button (2) on the left side of the working table. (section 5.4).

**Note:** if the working table is provided with optical fiber coupling, the button for Air Jet system activation is the same switch for On/Off optical fiber coupling light.

On the working tables with Blue Touch panel, use the Air Jet key (1).

**To activate/deactivate the Air Jet system on the Blue Touch panel:** use the On/Off Air Jet key (1) on the command panel. When Air Jet system is activated, the continuous air jet starts as soon as the handpiece is removed from the holder.

When pressing the propulsion/activation pedal, the system is interrupted, and the handpiece operates normally. When releasing the pedal, the air jet will restart until the system is turned Off or the handpiece is placed in its holder.

## 6.12 - X-Ray Viewer

**To turn On the x-ray viewer:** press the On/Off key (1) on the command panel.

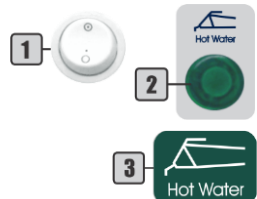
## 6.13 - Thermo Comfort System (Optional)

**To activate the Thermo Comfort:** press the button (1) on the right side of the working table (section 5.3) or cross flex command panel. The green LED (2) on the instrument panel will light up.

**To activate the on working tables with Blue Touch Panel:** use the Thermo Comfort key (3). Above the button, the green LED (2) will light to indicate that device is operating. (section 5.4).

When the device is started, wait for approximately 10 minutes for the initial heating of the water.

## 6.14 - Anti-Stress System (Optional)

**To activate/deactivate Anti-Stress system:** press the “On/Off” key (5).

**To enable the automatic massage system:** press the “AUT” key (3). The system will start executing 14 types of massage combinations, alternating one combination every minute.

**To toggle massage combinations:** use the “AJUSTES” keys (8) while performing automatic massage to advance or repeat the desired massage combination.

**To adjust the massage time:** press the “TEMPO” key (6) and then use the “AJUSTES” key (8) to adjust the duration of the massage (from 5 to 45 minutes).

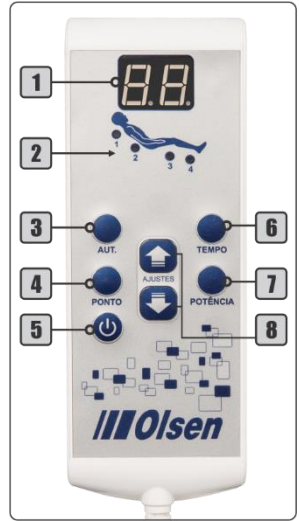
**To adjust the vibration power:** press the “POTÊNCIA” key (7) and then use the “AJUSTES” key (8) to adjust up to 10 levels of vibration intensity for the massager’s motors.

**To enable/disable massagers:** use the “PONTO” key (4) to define which massagers should work during the massage. Use the “AJUSTES” key (8) just after the “PONTO” key (4) to customize the running massages. The LED’s (2) indicated the motor that will be active.

**To restore factory settings:** press the “AUT” key (3) twice quickly.

**20 seconds after the last command executed, the display (1) will show the initials AU, but without affecting the programming in progress.**


## 6.15 - Monitor Holder (Optional)

The monitor holder features a rotation on the axis, tilt adjustment and lateral rotation for the monitor.

The monitor is not part of the monitor holder.

**Do not use on the monitor holder:**

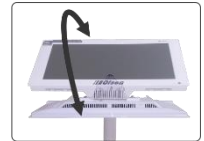- 1 - Equipment powered with more than 24 Volts;
- 2 - Equipment with weight more than 1 kg.


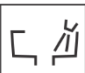
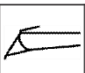
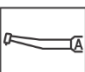
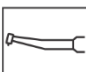
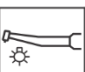
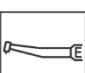
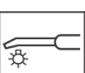
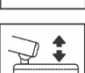
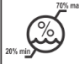
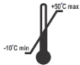
## 6.16 - Cart System (Optional)

See the *User Manual Dental Line*.

## 7 - Symbology

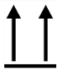The symbology complies with IEC 60601-1 and IEC 60878.

	Patient Chair Up (Dental)		Patient Chair Down (Dental)		Zero Position
	Backrest Up		Backrest Back		Work Position
	Bowl Flush		Cup Filler		Multifunction Syringe (Air-Water)
	Dental Air Motor		Turbine		Turbine with Illumination
	Dental Low Voltage Electric Motor		Ultrasonic Scaler		Curing Light
	Saliva Ejector		Suction Handpiece		Suction Handpiece with Hand Control Valve
	Variability, for Rotating Movement		Dental Operating Light		Lighting
	Pneumatic Brake		Hand Control Valve		Foot Switch
	Spray-cooling		"OFF" (Power)		"ON" (Part of the Equipment)
	Stop		"ON" (Power)		"OFF" (Part of the Equipment)
	Emergency Stop		"ON"/"OFF"		Write Data into Store (Selection)
	Manufacturer		Caution		Warning, Electricity
	General Warning Sign		Level		Sterilizable Up to the Temperature Specified
	Operating Instructions		General Prohibition Sign		Refer to the Instruction Manual
	General Mandatory Action Sign		Stepping Prohibited		Type BF Applied Part

	Type B Applied Part		Serial Number		Protective Earth (Ground)
	Earth (Ground)		Keep Away from Heat		Humidity Limitation
	Fragile, Handle with Care		Temperature Limitation		Keep Dry
	Heaping Up		This Side Up		Alternating Current
	Non-Sterile	 Authorized Representative in the European Community			

## 8 - Olsen Accredited Technical Assistance Network

To access the Olsen certified technical assistance network for installation and maintenance, contact us by e-mail [export3@olsen.odo.br](mailto:export3@olsen.odo.br) or by phone +55 48 2106 6000.

**The installation and maintenance of the equipment and its accessories must only be carried out by Olsen technicians.**

**Use Olsen genuine parts only. The use of non-genuine parts, accessories or components may compromise the proper functioning of the equipment.**

**Do not make adaptations, modifications or changes to the equipment, its components or accessories.**

## 9 - Warranty Terms

The warranty period is of 12 months starting from the equipment's installation date since the installation be made during the 90 days after the purchasing date and under the terms of this certificate.

- 1 - The maximum storage period must be 3 months from the purchasing date. In case of this period is exceeded, the warranty period will start counting even if the product is kept in storage.
- 2 - Upholstered parts are under warranty for 6 months.
- 3 - Operating light mirror, fuses, wires, and transformers are not covered by this warranty.
- 4 - The warranty is limited to the repair or replacement of defected parts and does not cover defects originated by:
  - a) Non-observance of the use and maintenance instructions;
  - b) Falls, crashes and inadequate storage;
  - c) Effects caused by the agents of nature;
  - d) Damage caused to the upholstered parts, due to improper use of chemical products, exposure to inadequate weather conditions, contact with fabrics, leather, disposable gloves, paints, pigmented detergents, cutting or piercing objects;
  - e) Damage to painted parts and plastic covers caused by improper use of chemical products or by contact with disposable gloves and cutting or piercing objects;
  - f) Connection with a wrong voltage power supply.
- 5 - This warranty will cease when:
  - a) The warranty period expires;
  - b) The user makes any changes to the product or uses inappropriate accessories;
  - c) For adulterations in the Warranty Certificate, purchase, installation or service document;
  - d) Installation and technical service is done by an unauthorized technician;
  - e) The equipment is stored for more than 6 months after the invoice issuing date;
  - f) For the use of spare parts other than Olsen originals.
- 6 - The repairing or replacement of parts under the warranty period will not extend its original expiring date.
- 7 - The expenses resulted from the equipment's installation, scheduled maintenance, travel and accommodation costs of the service staff involved in the calls for service for the equipment's installation or repair will run under the owner's responsibility and in accordance with the distributor's norms.
- 8 - The customer, after checking the installation, must keep the purchasing invoice and the equipment's serial number until the warranty period end, as in the case of service needed, the serial number, the invoice number, and purchasing or installation date will be required.
- 9 - On all calls for service during the warranty period the equipment's serial number and a copy of the purchasing or installation invoice must be shown. In the case of this information is not presented, the service will not be done under warranty.

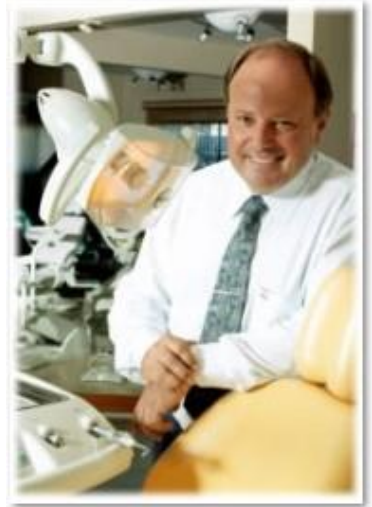
## 10 - Message from the President

**Olsen and its clients:  
A successful relationship.**

I have linked my name to the factory and to the Dental and Medical equipment that are currently produced and trade in more than 100 countries having in mind the responsibilities and long-term response to this initiative.

Our products are modern, innovative, durable and of low maintenance cost. These characteristics have been achieved thanks to our competent and dedicated team, which make me very proud for many reasons, as they are always giving the best of their creative capacity to our clients.

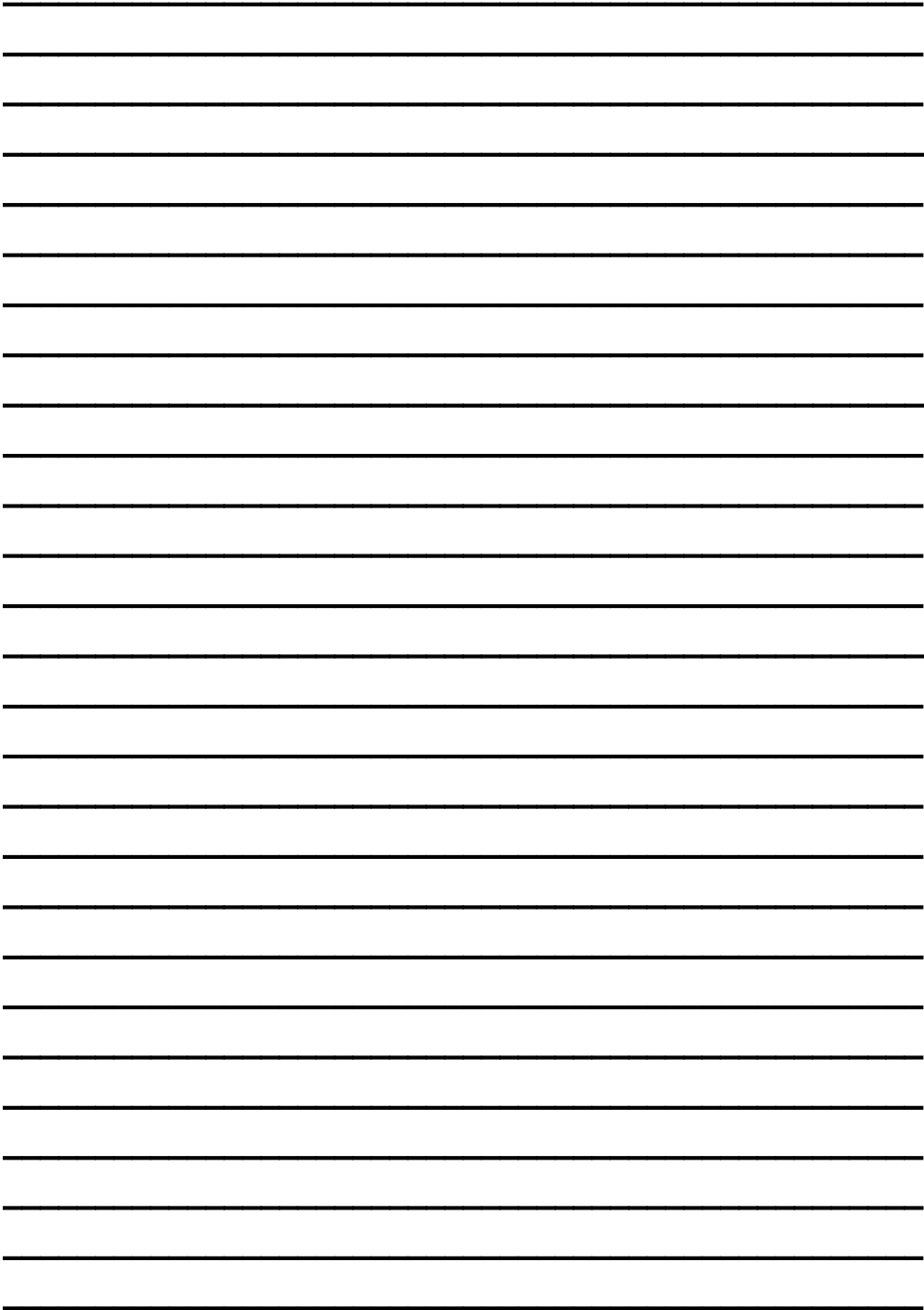
Our company will always be open to all those who prefer Olsen products, for any necessary information and technical assistance, but especially for comments regarding the relationship with customers. We expect this connection always brings you satisfaction, resulting in more and more benefits to all of us.

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